

# Positioning Checklist for the Rifton TRAM

Use this Positioning Checklist as a convenient way to ensure optimal use of the TRAM. Write notes to customize your instructions for each individual.

\_\_\_\_\_  
INDIVIDUAL'S NAME

\_\_\_\_\_  
STAFF NAME

\_\_\_\_\_  
DATE



## Frame

- Standard frame
- Low Base frame

Client uses hand grips on frame

- Yes  No



## Scale

\_\_\_\_\_ Individual's weight

\_\_\_\_\_ Target partial weight-bearing amount (in lbs)

## Body Support System



- Snug to trunk
- Loosely around trunk

## Height (in sitting position prior to transfer)



\_\_\_\_\_  
(Specify the number 1-10)

## Height (for standing position)



\_\_\_\_\_  
(Specify the number 1-10)

# Forearm Support Holder



## Forearm Support Mounting Bracket

- Longer end downward
- Longer end upward

---

### Low Position



#### Left

- Low and forward
- Low and backward

#### Height:

- ½ notch visible below
- no notches visible
- ½ notch visible above
- 1½ notches visible above
- 2½ notches visible above

#### Right

- Low and forward
- Low and backward

#### Height:

- ½ notch visible below
- no notches visible
- ½ notch visible above
- 1½ notches visible above
- 2½ notches visible above

### High Position



#### Left

- High and forward
- High and backward

#### Height:

- 2½ notches visible below
- 1½ notches visible below
- ½ notches visible below
- no notches visible
- ½ notch visible above

#### Right

- High and forward
- High and backward

#### Height:

- 2½ notches visible below
- 1½ notches visible below
- ½ notches visible below
- no notches visible
- ½ notch visible above

## Forearm Support



### Left

Angle up  Angle down  Straight

\_\_\_\_\_ Rotation position

(specify as o'clock when positioned in TRAM)

### Right

Angle up  Angle down  Straight

\_\_\_\_\_ Rotation position

(specify as o'clock when positioned in TRAM)

---

## Arm Platforms



### Left

Attachment position with platform FORWARD

Attachment position with platform BEHIND

Angle up  Angle down  Straight

\_\_\_\_\_ Rotation position

(specify as o'clock when positioned in TRAM)

### Right

Attachment position with platform FORWARD

Attachment position with platform BEHIND

Angle up  Angle down  Straight

\_\_\_\_\_ Rotation position

(specify as o'clock when positioned in TRAM)

## Used for Transfer

### Thigh Straps



- Narrow  Wide
- Use third seat strap  Narrow  Wide

### Position in TRAM

- Client faces caregiver handle
- Client faces away from caregiver handle

### Left

- Front "O" ring  
Yellow (recommended) Other: \_\_\_\_\_
- Back "O" ring  
 Red  White  Blue  Gray

### Right

- Front "O" ring  
Yellow (Recommended) Other: \_\_\_\_\_
- Back "O" ring  
 Red  White  Blue  Gray

---

## Used for Mobility

### Pelvic Support Small Medium Large



### Color-coded Clips

- Left front strap  
 Red  White  Blue  Gray

- Right front strap  
 Red  White  Blue  Gray

- Left back strap  
 Red  White  Blue  Gray

- Right back strap  
 Red  White  Blue  Gray

---

## Directional Locks



### Direction Lock Location: (Check all that apply)

- Caster near caregiver handle, on caregiver's right  Up  Down
- Caster near caregiver handle, on caregiver's left  Up  Down
- Caster located at frame end, on client's right  Up  Down
- Caster located at frame end, on client's left  Up  Down