

# Attendant Guide Bar

K589 Product Manual



## **WARNING**



- Thoroughly read and understand the information in this product manual before attempting to use this product. If the procedures and instructions in this product manual are not followed, serious injury or death could occur.
- A qualified professional must assess the appropriateness and safety of all equipment for each user.
- This product is intended for use by clients of unreliable judgment. Adult supervision is required at all times.
- To prevent falls and injuries:
  - Tighten all adjustment knobs before use and immediately after making any adjustments.
  - Adult supervision is required at all times. Always remove guide bar when not in use.
- To prevent structural failure, which may result in serious injury or death:
  - Immediately remove this product from use when any condition develops that might make operation unsafe.
  - Do not use Rifton components or products for any purpose other than their intended use.

## **User modifications**


### **WARNING**


To prevent serious injury or death, do not modify or alter Rifton products or components, or use Rifton products or components in conjunction with products from other manufacturers. Rifton does not accept responsibility for any modifications or alterations made to our components or products after they leave our premises. Customers modifying or altering our components or products, or using them in conjunction with products from other manufacturers, do so at their own risk.

## **Key for users**

Use this key to determine which sections of this product manual apply to you.

 **Technical Users** For professionals who order and set up Rifton products.

 **Home Users** For care-givers who use Rifton products on a regular basis.

 **Maintenance Personnel** For anyone who is responsible for service or re-ordering of Rifton products and parts.

## Guide bar

**⚠️ WARNING** To prevent injury, adult supervision is required at all times. Always remove guide bar when not in use.

### Attaching

1. For maximum stability attach the guide bar centered on the front of the top bar of the Pacer (see Figure 3a).
2. Guide bar is attached by tightening knob (A) to clamp it on.



Figure 3a

### Adjustments

See Figures 3a and 3b.

3. Rotate guide bar until it is behind the client for pushing or in front of the client for pulling by the caregiver.
4. Tighten knob (A) to make guide bar stay at the desired height even when the handle is released.



Figure 3b

## Warranty Statement

If a Rifton product breaks or fails in service during the first year, we will replace it free of charge.

### IMPORTANT

Please save this product manual for future reference. Additional copies are available at <http://www.rifton.com/customer-service/product-manuals>.

# Rifton Contact Information



## Email

[Sales@Rifton.com](mailto:Sales@Rifton.com)



## Phone

**800.571.8198**  
9–5 EST



## Mail

Rifton Equipment  
PO Box 260  
Rifton NY 12471–0260



## Fax

**800.865.4674**



## Online

[www.Rifton.com](http://www.Rifton.com)

## To order replacement parts

1. **Locate the ID number** of the product on the small white label.
2. Have this number available when you call **800.571.8198** for your customer service representative.

### **Use only replacement parts supplied by Rifton Equipment.**

We are glad to supply replacement parts. Although Rifton makes every effort to supply correct parts and instructions for repairing or refurbishing your equipment, you are responsible to make sure that the repairs or modifications are correctly and safely completed.



Find letters of medical necessity  
and informative articles at:  
**[www.rifton.com/pacer](http://www.rifton.com/pacer)**